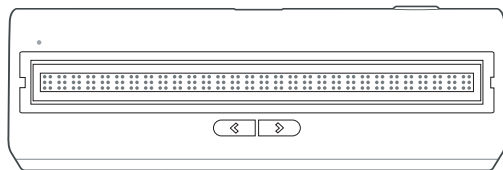
Portable Braille Display

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Dimensions: 209 × 75 × 28 mm

Braille Cells: 30 (single line)

Wireless Connectivity: Bluetooth (BLE 4.2–5.0)

Wired Connectivity: USB-C

Battery: Rechargeable lithium battery

03/Device Overview

Display Panel

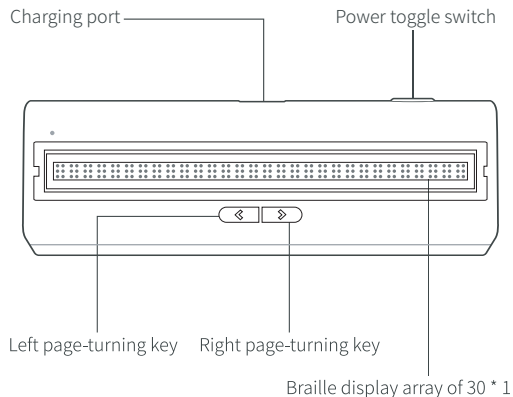
The display consists of 30 six-dot refreshable braille cells, capable of presenting dynamic braille content.

Control Buttons

Includes a power slide switch and up/down navigation buttons for device operation.

Charging Port

Equipped with a USB Type-C charging port supporting fast charging (approximately 1 hour).

04/Basic Functions

Braille Display

Displays text information through 30 refreshable 6-dot braille cells.

Connection & Compatibility

Supports seamless Bluetooth connection with computers, smartphones, and tablets.

Compatible with Windows, iOS, and Android systems.

Wujie Bookstore App

Provides access to an online braille bookstore.

Supports uploading e-books in TXT, Word, PDF, and other formats for braille conversion.

Supports braille translation for over 40 languages.

Supports the display of music scores, tactile graphics, diagrams, and formulas.

A community forum is available for users to share reading experiences and connect with others.

This product is compatible with screen reading software across multiple platforms including iOS, Android and Windows. For detailed instructions, please refer to the tutorial page on our official website: <https://www.wujieqh.com/tutorials.html>

Voice Assistant

The device plays a startup sound when powered on.

Bluetooth Notifications

Announces “Bluetooth connected” when successfully connected.

Announces “Bluetooth disconnected” when the connection is lost.

Voice Wake-Up

Wake word: “Hello Zhichi” (voice assistant name)

After activation, wait 1–2 seconds before issuing commands.

Voice Commands

Check battery / volume:

“Check battery” / “Check volume”

Adjust volume:

“Increase volume” / “Decrease volume”

(5 levels, from 20 to 100 in steps of 20)

Mute mode:

“Turn on mute mode” / “Turn off mute mode”

Fixed volume:

“Set to maximum volume (100)”

“Set to minimum volume (20)”

“Set to medium volume (60)”

Physical Controls

Mute: Long press both page buttons to toggle mute mode

Charging prompts:

“Start charging” when plugged in

“Stop charging” when unplugged

Battery alerts:

“Battery full” when fully charged

Low battery alerts at 20%, 10%, and 5% remaining.

05/Operation Instructions

Ensure Bluetooth is enabled on your phone or computer.

Slide the power switch to turn on the device.

The device will automatically enter Bluetooth pairing mode.

First-Time Connection

Open the braille display settings in the Wujie Bookstore App.

Search and connect to the device with the default name: **BDF-30D1F15V**

After pairing, the app will display device information such as model and battery level.

06/Precautions

Do not remove the protective film on the braille display surface.

Avoid applying excessive force while reading.

Keep the device away from sharp objects.

If Connection Fails

Ensure the device is powered on and has sufficient battery. Keep within Bluetooth range (within 5 meters).

Turn Bluetooth off and on again on both devices, then reconnect.

Durability

Withstands drops of up to 1.5 meters.

Avoid severe impacts or dropping from greater heights.

Environment

Do not use or store in high temperatures (>40°C) or high humidity (>80%).

Keep away from fire and corrosive substances.

Notes when Reading

Only touch the display after the braille refresh is complete.

Touching during refresh may cause irregular braille dots.

If braille dots are not fully displayed:

Press both page buttons simultaneously to refresh.

After-Sales Service

One-year warranty provided.

For support:

Phone: +86 19030916561

Email: support@wujieqh.com

Product Information

Product Name: "Fangcun" Portable Braille Display

Model: BDF-30D1F15V

Standard: GB/T 41182-2021

Manufacturer: Chongqing Infinite Horizons Co., Ltd

Website: www.wujieqh.com

Address: Building 1, No. 21 Zhuoyue Road, Longxing

Town, Liangjiang New Area, Chongqing, China

WARNING

Do not disassemble, impact, squeeze, or heat the battery.

Avoid high temperatures, direct sunlight, and prolonged charging.

Stop using immediately if swelling, deformation, unusual noise, or overheating occurs.

Do not use the device if it has been submerged in water.

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